

dermadrive™

Dermatome skin grafting system

GD SERIES




deSouther
MEDICAL

GD PRECISION SKIN GRAFTING SYSTEM

The new slimline and ergonomically designed *GD Series* dermatome from De Soutter Medical produces consistent, uniform skin grafts time after time. The smooth high speed cutting action ensures an almost effortless and vibration free procedure. The GD-113 dermatome head is designed to be used with the CM2 controller system. Alternatively, the head with its integral E coupling allows it to be used with other De Soutter Medical handpieces, for instance, EPV pneumatic and ECV electric motors. Please ask for further details.



Cut thicknesses are easily selected and constantly maintained using the calibrated depth gauge. Skin grafts from 0.05 mm to 1.00 mm are adjustable in 0.1 mm increments.

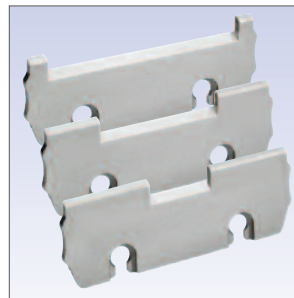

Accurately controlled graft widths of 73 mm, 48 mm and 23 mm are readily achieved by fitting the appropriate easy-fit sole plate.



Stericut precision ground, double facet, ultra sharp blades provide accurate and consistent cutting performance whilst permitting very thin skin grafting. Blades are supplied sterile packed (5 per box). The GD-113 dermatome head enables quick and safe blade replacement.



SYSTEM SPECIFICATION

DERMATOME SPECIFICATION

	Cutting width mm	Cutting thickness range mm	Operating speed range rpm	Weight kg
	73	0.05 - 1.00	2000 - 15000	0.28
The dermatome head is supplied with a 73 mm sole plate as standard. Optional sole plates: 48 mm cutting width - Order code 17260 23 mm cutting width - Order code 17340				

MICRO MOTOR AND POWER SUPPLY

The dermatome head fits quickly onto the universal micro motor which is powered by the variable speed CM2 controller. The motor can be controlled either by the footswitch or directly from the controller itself.

Please specify voltage and plug configuration when ordering.

Power supply specification

Input voltage: 230/115/100 50-60Hz
Power: 60VA
Dimensions: 181 x 101 x 119 mm
Weight: 1.79/0.12 kg controller/motor



STERILISATION CONTAINER

This purpose designed compatible delivery system will ensure your instruments are maintained in good condition at all times. The container is 100% sterilisable by steam, dry heat ('flash') to 160C, ethylene oxide (ETO) or gamma irradiation.



HOW TO ORDER

System components	Order code
GD-113 head	17200
CM2 motor controller, micro motor and foot pedal - Universal	1292334
Blades (pack of 5)	S11-103
Sterilisation container	8570
Complete system ordering code	GDA-113/CM2

Mains power cordsets 3 metres long.
Please specify - Order code

UK	8290
Euro	8300
US/Canada	8310
Australia	8320
New Zealand	8320
Swiss	14700

deSoutter
MEDICAL

***For further information on De Soutter Medical
powered instrument and cast saw systems
please contact the address in your area***

UNITED KINGDOM (INTERNATIONAL HEAD OFFICE)

De Soutter Medical Limited
Halton Brook Business Park, Weston Road, Aston Clinton, Aylesbury, Bucks HP22 5WF
Tel: +44 (0) 1296 634 000 Fax: +44 (0) 1296 634 033
Email: info@de-soutter.com

AUSTRALIA

De Soutter Medical Australia Pty Ltd
2/12-14 Apollo Drive, Hallam, Victoria 3803
Tel: +61 (0) 3 9702 4441 Fax: +61 (0) 3 9702 4484
Email: australia@de-soutter.com

AUSTRIA

De Soutter Medical Austria
Zweigniederlassung Österreich, Dietrichsteingasse 10, A-3400 Klosterneuburg
Tel: +43 (0) 676 96 71 770 Fax: +43 (0) 2243 21 656
Email: austria@de-soutter.com

BELGIUM

De Soutter Medical Belgium
In De Bruel 30, 3620 Lanaken
Tel: +32 (0) 89/47 15 37 Fax: +32 (0) 89/70 12 19
Email: belgium@de-soutter.com

FRANCE

De Soutter Medical France
949 Avenue Parc des Expositions, 33260 La Teste de Buch
Tel: +33 (0) 5 56 54 89 36 Fax: +33 (0) 9 70 61 37 60
Email: france@de-soutter.com

GERMANY

De Soutter Medical Deutschland
Bahnhofstraße 4, D-66625 Nohfelden
Tel: +49 (0) 68 52-99 12 46 Fax: +49 (0) 68 52-99 12 47
Email: deutschland@de-soutter.com

ITALY

De Soutter Medical Italia
Localita' Fornace SNC, 27022 Casorate Primo - PV
Tel: +39 (0) 2 9009 4098 Fax: +39 (0) 2 9009 2673
Email: italy@de-soutter.com

NETHERLANDS

De Soutter Medical Nederland
Gelderlandhaven 2X, 3433PG Nieuwegein
Tel: +31 (0) 85 0491480 Fax: +31 (0) 85 0491489
Email: nederland@de-soutter.com

UNITED STATES OF AMERICA

De Soutter Medical USA, Inc.
224 Rolling Hill Road, Suite 12A, Mooresville, NC 28117
Tel: +1 (704) 655 9040 Fax: +1 (704) 987 2035
Email: usa@de-soutter.com

www.de-soutter.com

120-EN-7